

Release Notes **Tobii Dynavox Communicator 5**

Introduction

Tobii Dynavox Communicator 5 is a software package designed to help individuals with communication disabilities communicate more efficiently. Communicator 5 converts text and symbols into clear speech, and offers easy-to-use tools for computer access, e-mail, text messaging, telephony, environmental control and more.

System Requirements

- Windows 7, Windows 8, Windows 8.1 and Windows 10
- 2.0 GB RAM
- 3.0 GB Hard Drive
- 1.3 Ghz processor

Customer support: For more information or support, please visit www.TobiiDynavox.com.

Release Version 5.3.1

Release date: 2016-12-09

Improvements

- Saving page sets to a non C Drive location failed (TDCOM-565)
- Possible to get trapped in Content Products if set as Startup Page (TDCOM-566)

Release Version 5.3

Release date: 2016-11-30

New Features

- You can now add and launch Files from the Home Page.
- The Voice Pitch can be adjusted for Acapela Voices, which allows you to adjust the age of children's voices.
- Scanning Windows Control had been updated to provide better contrast on all backgrounds.
- Communicator can now import messages from the online Message Banking Service.
- You can use Beam to send SMS on Android phones on any Windows 10 PC.
- New Keyboard Layouts and a new High-Contrast color scheme.

Improvements

- Licensing was not functioning when the device was offline (TSAR-217)
- Improved saving times (CVI-647)
- Fixed bugs related to losing focus in Windows 10 Tablet mode (CVI-589)

Release Version 5.2.2

Release date: 2016-10-31

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com or www.myTobiiDynavox.com.

Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

Improvements

- In Sono Flex, moving between pages will cause the message window to speak (TDCOM-537)



Only for English, Danish and German

Release Version 5.2.1

Release date: 2016-08-29

Improvements

- In Sono Flex, when 2 words on a label, speak as you type sound setting does not work. (TDCOM-515)



Only for English, Danish and German

Release Version 5.2

Release date: 2016-04-25

New Features

- Change the Dwell Free Keyboard from a Beta Feature to a full Communicator Keyboard
- Dwell-Free has been translated into German
- Dwell-Free can be set as a Communicator Keyboard
- The Dwell-Free prediction engine will learn new words
- The error correction mode in Dwell-Free provides Swiftkey word predictions as you type
- Dwell-Free has been improved to allow for long text editing, with improved cursor movement options and the ability to insert new sentences to the location of the cursor
- Dwell-Free now has auto-correct features, such as intelligent spacing for punctuation and auto-capitalization

Improvements

- Crash when duplicating non-button objects (CVI-548)
- Japanese Window Keyboards cannot turn IME on/off (CVI-484)
- Japanese Large Keys Keyboard displays incorrect second page for "な" (CVI-483)
- Italian abc Keyboard layout is missing àùìè (CVI-475)
- Activating Communicator Fails is the Machine Name contains non-English characters (CVI-434)
- In Sono Key navigating to the Alarm page clears the Message Window (CVI-411)

Release Version 5.1

Release date: 2015-12-09

New Features

- The new Abbreviation Expansion feature allows users to access their phrases faster than ever.
- There are new Alarm and Go To Desktop page sets

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com or www.myTobiiDynavox.com.

Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

- The user can set a Different voice to be used for speech output and feedback sounds
- The Combine 2 IR commands action has been updated to allow the user to also Combine 3 IR commands
- Added Keyboard support for Croatian, Czech, Hungarian, Polish, Slovak, Slovenian, & Ukrainian
- Communicator 5 is now available in Finnish, Brazilian Portuguese, & Japanese

Improvements

- Communicator leaks memory when loading cddx files (TDCOM-327)
- Links refer to www.pagesetcentral.com (TDCOM-311)
- Importing TC4 E-Mail into C5 may cause a crash on startup (TDCOM-256)
- Unexpected results when injected page has the same name as a page in the main page set (TDCOM-193)
- Skype OSK Page Set cannot place calls (TDCOM-294)

Release Version 5.0.1

Release date: 2015-09-22

Improvements

- Improved reliability when sending or receiving text messages through a Bluetooth phone (CFIVE-6573)
- Updated Facebook to use the new API 2.0 (CVI-17)
- Occasionally unable to Save Communicator 4 Page Set in Communicator 5 (TDCOM-258)

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com or www.myTobiiDynavox.com.

Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact