tobiidynavox

Release Notes

Compass Software, Version 2.2.0

New Features and Enhancements for Compass Software

- Acapela voices, in addition to the already supplied IVONA voices, are now available for new I-Series and T-Series devices running Compass. Existing T-Series and App customers will be able to purchase the voices through <u>myTobiiDyna-vox.com</u>
- Native voices and other installed text-to-speech engines are now available to the Compass software. Note that on T-Series devices these voices are only available on Open (non-dedicated) devices.
- IR Learning is now available on all I-Series and T-Series devices.
- Swedish and Norwegian versions of the Compass software and some content (All Access and Navigator pagesets) are now available.
- The default configuration of the User Interface and on-screen keyboards are now initialized based on the language settings of the device operating system (iOS and Android).
- Images will now be scaled to the size of the button (or other object) when added. This improves page load speed and reduces the file size of the content.
- The "Eye Tracking" Access Method has been renamed "Gaze Interaction".
- There are several updates to reflect the change of the company name to Tobii Dynavox.

New Features and Enhancements for myTobiiDynavox.com

- The website "mydynavox.com" is now "myTobiiDynavox.com". The former mydynavox.com URL is still live and will redirect users to myTobiiDynavox.com.
- The Acapela voices and third party content, such as the PODD and Gateway pagesets, are now available for purchase through myTobiiDynavox.com.

Copyright CTobii Technology AB. Illustrations and specifications do not necessarily apply to products and services offered in each local market. Technical specifications are subject to change without prior notice. All other trademarks are the property of their respective

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.tobiidynavox.com or www.myTobiiDynavox.com.

Contact Your Reseller

For questions or problems with your product, contact Customer Support at your Tobii Dynavox Reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For our Resellers contact details, see <u>www.tobiidynavox.com</u>