

Release Notes **Compass Software**

Release Version 2.4.1

Release date: 2015-09-22

Improvements for Compass Software

- Corrected a problem with Scanning with Audio Cue where Audio Cue would stop or slow down after extended use.
- Improvements to Bluetooth switch performance with the iOS app.
- Incorrect labeling of the RJ Cooper BT Switch options has been corrected.
- Improved character prediction in Danish and German.
- In previous versions of Compass, permission to use third party content (such as PODD) could be lost when WIFI connection was dropped. This issue has been fixed.

New Features and Enhancements for Compass Software

- French version of the Compass software and some content are now available.
- Users are now able to use the Symbol Manager to install symbol sets purchased through myTobiiDynavox.
- This release includes the brand new US English child voice, Justin. Download the Justin child voice through the Manage Voices menu.
- Compass 2.4 comes with an updated text-to-speech engine, updated Ivona voices, and ten all new Ivona voices. The new text-to-speech engine is not backwards compatible with older voices, so updated Ivona voices must be downloaded through the Manage Voices menu. If you are using an Ivona voice as your primary voice or Audio output voice, that particular updated voice will automatically download and install.

New Ivona voices by language:

- da-DK: Mads, Naja
- en-US: Justin (child voice)
- es-ES: Conchita, Enrique
- nb-NO: Liv
- nl-NL: Lotte, Ruben
- pt-BR (only available on myTobiiDynavox): Ricardo, Vitoria

Release Version 2.3

Release date: 2015-06-29

Improvements for Compass Software

- Fixed an issue where black borders were appearing on images added to buttons.
- For the Scanning Access Method with an Audio cue or Zoom, the issue with highlight timing falling behind has been fixed.
- The issue has been fixed where Access Methods stopped working when a label on a page included a variable such as date or time.
- Fixed the issue in the Scanning Access Method set to 2 Switch Scanning where scanning would stop working on some popups and system menu items.

Support for Your Tobii Dynavox Device

Get Help Online

See the [product-specific Support page](#) for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com or www.myTobiiDynavox.com.

Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

New Features and Enhancements for Compass Software

- Dutch and Danish versions of the Compass software and some content are now available.
- Option added to stop scanning after X number of passes. Options: Infinite, One, Two, Three, Four.
- The Gaze Selection menu is now suppressed when using Compass on an I-Series.

Release Version 2.2.1

Release date: 2015-04-09

Improvements for Compass Software

- Fixed issue with scanning where multiple selections were made if the switch was not released.
- Resolved issue where audio cues were continuing to play while the device was asleep. Audio cues now stop while device is in sleep mode and resume when you wake the device.

New Features and Enhancements for myTobiiDynavox.com

- Full site update, redesign, and rebranding.
- Addition of a store section where pagesets and voices are available for purchase.
- Single sign in for the Community area.

Release Version 2.2.0

Release date: 2015-02-18

New Features and Enhancements for Compass Software

- Acapela voices, in addition to the already supplied IVONA voices, are now available for new I-Series and T-Series devices running Compass. Existing T-Series and App customers will be able to purchase the voices through myTobiiDynavox.com
- Native voices and other installed text-to-speech engines are now available to the Compass software. Note that on T-Series devices these voices are only available on Open (non-dedicated) devices.
- IR Learning is now available on all I-Series and T-Series devices. The T10 requires the IR and Switch Adapter accessory.
- Swedish and Norwegian versions of the Compass software and some content (All Access and Navigator pagesets) are now available.
- The default configuration of the User Interface and on-screen keyboards are now initialized based on the language settings of the device operating system (iOS and Android).
- Images will now be scaled to the size of the button (or other object) when added. This improves page load speed and reduces the file size of the content.
- The “Eye Tracking” Access Method has been renamed “Gaze Interaction”.
- There are several updates to reflect the change of the company name to Tobii Dynavox.

New Features and Enhancements for myTobiiDynavox.com

- The website “mydynavox.com” is now “myTobiiDynavox.com”. The former mydynavox.com URL is still live and will redirect users to myTobiiDynavox.com.
- The Acapela voices and third party content, such as the PODD and Gateway pagesets, are now available for purchase through myTobiiDynavox.com.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com or www.myTobiiDynavox.com.

Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact