

## Release Notes

### Compass Content

#### Version 2.2.1

#### Master Page Pageset (English only)

- There is now only a single Master Page Pageset, as opposed to separate pagesets for each age range. With the addition of the Topic Picker (see below), age-appropriate topics can easily be selected. Message and vocabulary customizations are preserved across layout changes.
- Topics
  - Added Topic Picker/Creator. Access through Levels (Dashboard > Levels > Topics > Select Topics from Library)
    - Disable Alphabetize Topics if you would like to reorder topics on the master page using the Editor.
  - Topic pages can now toggle between Visual Scene and Grid layouts, preserving changes to messages. Access through Dashboard>Levels>Topics>Page Layout.
  - Added 11 new topics: Advocating, Arguing, Asking for Help, Crimes, Dating, Directing My Care, Job Coach, My Feelings, Planning, Presentation and Sexuality
- Supports
  - Social Narrative supports are now editable via the Modify mode. Minor Modify mode editing improvements made to other supports.
- Word Lists
  - Access to separate word list for different ability levels: (levels 1 & 2), context dependent (levels 3 & 4) & independent (Level 5).
  - Change the word lists for a specific level in Settings (Settings > Interface > Age & Ability Level > Ability Level).
  - Note: Customization to words in one ability level will NOT transfer to the others.
  - All word list vocabulary is on a single page to make editing easier.
  - Sexuality words can be enabled/disabled through Levels (Levels > Word Lists > Show Sexuality Words).
- Keyboards
  - Added the non grid-aligned AEIOU keyboard.
  - Added support for Next Letter Prediction
  - Added Phrase Prediction
- Levels
  - Accessible through the Dashboard (see below) or the Settings tool.
  - Level Buttons change the grid layout and ability level for the pageset.
    - Level 1 - 9 button (3x3)
    - Level 2 - 16 button (4x4)
    - Level 2 - 25 button (5x5)
    - Level 2 - 36 button (6x6)

#### Support for Your Tobii Dynavox Device

##### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

##### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)

- Level 2 - 49 button (7x7)
- Changing the level will reset all level settings, except the page layout, back to the defaults.
- Added the Dashboard and related apps and features (Accessible from the Master page)
  - Photo Albums
  - Video Models
  - Message History
  - Rating Scale
  - Games: Target practice, Wack- A- Mole, Concentration, Matching, Word Scramble
  - Whiteboard
  - Texting
  - Remotes
  - Apps
  - Calculator - Basic math functions, +, -, X, /, =/-, C, backspace (for current number entry)
  - Volume controls
  - Mute toggle
  - Levels
- Added Support for Send To - Send messages from the Keyboard, Message History or calculator to a Topic or QuickPhrase button or send to Apps.

## Text-Based Pageset (English only)

- Main page can now be set to either topics or the keyboard in Setup Wizard
- Pageset layout is now 7x10 grid size instead of 6x10
- Message History button added to the toolbar
  - View recently spoken messages
  - Delete an individual message
  - Send the message to QuickFires, QuickPhrases or the last topic to save them in any message button
  - Copy to the message window in the keyboard
- Keyboard
  - All keyboard layouts are now grid-aligned for new 7x10 keyguard
  - Added phrase prediction
    - Populated from a set of starter phrases and phrase builders, plus message history
  - Added Send To button so messages can be saved to QuickFires, QuickPhrases or the last topic
- QuickFires - Uses a single page of phrases (Removed the "Casual" and "Formal" options)

## All Access - español (Spanish only)

- Pageset has been renamed "All Access - Español"
- Help and intro videos have been translated into Spanish
- Fixed issue with hiding/showing of Support, Scripts, & Topic Words buttons on Topic pages

## Support for Your Tobii Dynavox Device

### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)

## Version 2.2

### All Access (English, Spanish, Swedish, and Norwegian)

- Bug fixes
- Pageset is now available in Swedish and Norwegian.

### NavBar Pageset (English only)

- Levels
  - Levels button has been removed from the NavBar, and added to the new Dashboard page (see below).
- Topics
  - Message grid
    - Message buttons can now be edited using Modify mode.
    - Message grid uses swipable pages. Select the grid and use the swipe controller to access additional pages.
  - Topic pages can now toggle between Visual Scene and Grid layouts, preserving changes to messages. This is accessed through Dashboard>Levels>Topics>Page Layout.
  - Option to Enable/Disable Topic Words button or remove Topics from toolbar has been added to the Levels>Topics settings.
  - The Vocabulary database (Settings>Vocabulary) is no longer being used for Topic messages. Changes made to Topic messages in the Vocabulary database will NOT be reflected on topic pages.
  - Added 11 new topics: Advocating, Arguing, Asking for Help, Crimes, Dating, Directing My Care, Job Coach, My Feelings, Planning, Presentation and Sexuality
- Supports
  - Changes were made to various support to improve compatibility with the Modify mode. Note: The Social Narrative supports are not editable via the Modify mode.
  - Scripts
    - Scripts now swipe if there are multiple pages. Select the grid and use the swipe controller to edit additional pages.
- QuickPhrases
  - Message buttons can now be edited using Modify mode.
  - QuickPhrases Grid Layout is now independent of the Topic Grid Layout. Access through Dashboard>Levels>-QuickPhrases>Grid Layout
  - My Phrases tab is now visible at all levels
- Word Lists
  - Folders for sub-word lists are now at the end of the current word list, instead of the beginning.
- Core
  - Core Pages have been significantly revised.
    - Sub-categories of core ("verbs", "adjectives", etc) have been removed
    - Core Pages now have only a Main Tab for all the core words, and a Personal tab for any personalized words the user wants to add.

### Support for Your Tobii Dynavox Device

#### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

#### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)

- 7 Grid Layouts are available: 4x5, 5x6, 6x7, 7x8, 8x10, 10x10, 12x13
  - At 4x5 layout, you have a choice of how many words to expose (6/9/12/20).
  - At higher layouts, all words are shown.
- Color coding option has been added
- Keyboards
  - Added phrase prediction button to all of the prediction keyboards
  - Added "AEIOU" (without word prediction) option
  - Added Next Letter Prediction toggle button to all keyboards
- Levels menus
  - Added General setting dialog, which contains settings that apply to all across all the different types of pages
  - New dialogues added for Dashboard and Message History
- A new Dashboard page has been added to the toolbar. This contains some of the functions that were previous located elsewhere, as well as some new functions. Items found in the Dashboard are:
- New Items:
  - Message History – shows past messages. Includes only novel sentences constructed in Core or Keyboard pages.
  - Games – these are set of games provided for fun and for practice targeting buttons.
  - Calculator – a simple calculator for basic arithmetic functions.
- Items Re-located to Dashboard:
  - Photo Albums – these replace the Photo Stories, and have an updated appearance.
  - Video Models
  - Rating Scale
  - Whiteboard
  - Remotes
  - Apps
  - Volume controls
  - Mute
  - Levels
- Send To button added to most message windows. Allows send of message window text to another page to be stored on a button, to the Keyboard page to be edited, or to an outside App.
- Slot Picker page was revised

## Stroke & Brain Injury Persona Pageset (English only)

- Levels
  - Levels button has been removed from the Navbar, and added to the new Dashboard page (see below).
- Topics
  - Message grid
    - Message buttons can now be edited using Modify mode.
    - Message grid uses swipable pages. Select the grid and use the swipe controller to access additional pages.

### Support for Your Tobii Dynavox Device

#### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

#### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)

- Option to Enable/Disable Topic Words button or remove Topics from toolbar has been added to the Levels>Topics settings.
- The Vocabulary database (Settings>Vocabulary) is no longer being used for Topic messages. Changes made to Topic messages in the Vocabulary database will NOT be reflected on topic pages.
- Added 9 new topics: Advocating , Arguing/Negotiations , Asking for Help , Choices , Crimes , Dating , Planning , Presentations , Football
- QuickPhrases
  - Message buttons can now be edited using Modify mode.
  - QuickPhrases Grid Layout is now independent of the Topic Grid Layout. Access through Dashboard>Levels>-QuickPhrases>Grid Layout
  - My Phrases tab is now visible at all levels
- Word Lists
  - Folders for sub-word lists are now at the end of the current word list, instead of the beginning.
- Keyboards
  - Added phrase prediction button to all of the prediction keyboards
  - Added "AEIOU" (without word prediction) option
  - Added Next Letter Prediction toggle button to all keyboards
- Levels menus
  - Added General setting dialog, which contains settings that apply to all across all the different types of pages
  - New dialogues added for Dashboard and Message History
- A new Dashboard page has been added to the toolbar. This contains some of the functions that were previous located elsewhere, as well as some new functions. Items found in the Dashboard are:
- New Items:
  - Message History – shows past messages. Includes only novel sentences constructed in Core or Keyboard pages.
  - Games – these are set of games provided for fun and for practice targeting buttons.
  - Calculator – a simple calculator for basic arithmetic functions.
- Items Re-located to Dashboard:
  - Photo Albums – these replace the Photo Stories, and have an updated appearance.
  - Rating Scale
  - Whiteboard
  - Remotes
  - Apps
  - Volume controls
  - Mute
  - Levels
- Send To button added to most message windows. Allows send of message window text to another page to be stored on a button, to the Keyboard page to be edited, or to an outside App.
- Slot Picker page was revised

## Support for Your Tobii Dynavox Device

### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)

## Navigator Pageset (English, Spanish, Swedish, and Norwegian)

Pageset is now available in Swedish and Norwegian.

## Text Based Pageset (English only)

- Added a new column of buttons to the pageset. Now 7x10 instead of 6x10
- Added Message History button to the toolbar
  - Grid-aligned
  - View recently spoken messages
  - You can do the following with messages in the message history:
    - Delete an individual message
    - Send the message to QuickFires, QuickPhrases or the last topic to save them in any message button
    - Copy to the message window in the keyboard
- Keyboard changes:
  - QWERTY keyboard now grid-aligned for new 7x10 keyguard
  - Added phrase prediction and a set of starter phrases and phrase builders
    - Phrase prediction will also use phrase in the message history
  - Added Send To button so messages can be saved to QuickFires, QuickPhrases or the last topic
- QuickFires - Removed the “Casual” and “Formal” pages of phrases
- Updated new page templates to match the 7x10 layouts

### Support for Your Tobii Dynavox Device

#### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

#### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)