

Release Notes

Compass 2.2 Content

All Access (English, Spanish, Swedish, and Norwegian)

- Bug fixes
- Pageset is now available in Swedish and Norwegian.

NavBar Pageset (English only)

- Levels
 - Levels button has been removed from the NavBar, and added to the new Dashboard page (see below).
- Topics
 - Message grid
 - Message buttons can now be edited using Modify mode.
 - Message grid uses swipable pages. Select the grid and use the swipe controller to access additional pages.
 - Topic pages can now toggle between Visual Scene and Grid layouts, preserving changes to messages. This is accessed through Dashboard>Levels>Topics>Page Layout.
 - Option to Enable/Disable Topic Words button or remove Topics from toolbar has been added to the Levels>Topics settings.
 - The Vocabulary database (Settings>Vocabulary) is no longer being used for Topic messages. Changes made to Topic messages in the Vocabulary database will NOT be reflected on topic pages.
 - Added 11 new topics: Advocating, Arguing, Asking for Help, Crimes, Dating, Directing My Care, Job Coach, My Feelings, Planning, Presentation and Sexuality
- Supports
 - Changes were made to various support to improve compatibility with the Modify mode. Note: The Social Narrative supports are not editable via the Modify mode.
 - Scripts
 - Scripts now swipe if there are multiple pages. Select the grid and use the swipe controller to edit additional pages.
- QuickPhrases
 - Message buttons can now be edited using Modify mode.
 - QuickPhrases Grid Layout is now independent of the Topic Grid Layout. Access through Dashboard>Levels>-QuickPhrases>Grid Layout
 - My Phrases tab is now visible at all levels
- Word Lists
 - Folders for sub-word lists are now at the end of the current word list, instead of the beginning.
- Core
 - Core Pages have been significantly revised.
 - Sub-categories of core ("verbs", "adjectives", etc) have been removed

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com or www.myTobiiDynavox.com.

Contact Your Reseller

For questions or problems with your product, contact Customer Support at your Tobii Dynavox Reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For our Resellers contact details, see www.TobiiDynavox.com

- Core Pages now have only a Main Tab for all the core words, and a Personal tab for any personalized words the user wants to add.
- 7 Grid Layouts are available: 4x5, 5x6, 6x7, 7x8, 8x10, 10x10, 12x13
 - At 4x5 layout, you have a choice of how many words to expose (6/9/12/20).
 - At higher layouts, all words are shown.
- Color coding option has been added
- Keyboards
 - Added phrase prediction button to all of the prediction keyboards
 - Added "AEIOU" (without word prediction) option
 - Added Next Letter Prediction toggle button to all keyboards
- Levels menus
 - Added General setting dialog, which contains settings that apply to all across all the different types of pages
 - New dialogues added for Dashboard and Message History
- A new Dashboard page has been added to the toolbar. This contains some of the functions that were previous located elsewhere, as well as some new functions. Items found in the Dashboard are:
- New Items:
 - Message History – shows past messages. Includes only novel sentences constructed in Core or Keyboard pages.
 - Games – these are set of games provided for fun and for practice targeting buttons.
 - Calculator – a simple calculator for basic arithmetic functions.
- Items Re-located to Dashboard:
 - Photo Albums – these replace the Photo Stories, and have an updated appearance.
 - Video Models
 - Rating Scale
 - Whiteboard
 - Remotes
 - Apps
 - Volume controls
 - Mute
 - Levels
- Send To button added to most message windows. Allows send of message window text to another page to be stored on a button, to the Keyboard page to be edited, or to an outside App.
- Slot Picker page was revised

Stroke & Brain Injury Persona Pageset (English only)

- Levels
 - Levels button has been removed from the Navbar, and added to the new Dashboard page (see below).
- Topics
 - Message grid
 - Message buttons can now be edited using Modify mode.

Copyright ©Tobii AB. Illustrations and specifications do not necessarily apply to products and services offered in each local market. Technical specifications are subject to change without prior notice. All other trademarks are the property of their respective owners.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com or www.myTobiiDynavox.com.

Contact Your Reseller

For questions or problems with your product, contact Customer Support at your Tobii Dynavox Reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For our Resellers contact details, see www.TobiiDynavox.com

- Message grid uses swipable pages. Select the grid and use the swipe controller to access additional pages.
- Topic pages can now toggle between Visual Scene and Grid layouts, preserving changes to messages. This is accessed through Dashboard>Levels>Topics>Page Layout.
- Option to Enable/Disable Topic Words button or remove Topics from toolbar has been added to the Levels>Topics settings.
- The Vocabulary database (Settings>Vocabulary) is no longer being used for Topic messages. Changes made to Topic messages in the Vocabulary database will NOT be reflected on topic pages.
- Added 9 new topics: Advocating , Arguing/Negotiations , Asking for Help , Choices , Crimes , Dating , Planning , Presentations , Football
- QuickPhrases
 - Message buttons can now be edited using Modify mode.
 - QuickPhrases Grid Layout is now independent of the Topic Grid Layout. Access through Dashboard>Levels>-QuickPhrases>Grid Layout
 - My Phrases tab is now visible at all levels
- Word Lists
 - Folders for sub-word lists are now at the end of the current word list, instead of the beginning.
- Keyboards
 - Added phrase prediction button to all of the prediction keyboards
 - Added "AEIOU" (without word prediction) option
 - Added Next Letter Prediction toggle button to all keyboards
- Levels menus
 - Added General setting dialog, which contains settings that apply to all across all the different types of pages
 - New dialogues added for Dashboard and Message History
- A new Dashboard page has been added to the toolbar. This contains some of the functions that were previous located elsewhere, as well as some new functions. Items found in the Dashboard are:
- New Items:
 - Message History – shows past messages. Includes only novel sentences constructed in Core or Keyboard pages.
 - Games – these are set of games provided for fun and for practice targeting buttons.
 - Calculator – a simple calculator for basic arithmetic functions.
- Items Re-located to Dashboard:
 - Photo Albums – these replace the Photo Stories, and have an updated appearance.
 - Rating Scale
 - Whiteboard
 - Remotes
 - Apps
 - Volume controls
 - Mute
 - Levels
- Send To button added to most message windows. Allows send of message window text to another page to be stored on a button, to the Keyboard page to be edited, or to an outside App.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com or www.myTobiiDynavox.com.

Contact Your Reseller

For questions or problems with your product, contact Customer Support at your Tobii Dynavox Reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For our Resellers contact details, see www.TobiiDynavox.com

- Slot Picker page was revised

Navigator Pageset (English, Spanish, Swedish, and Norwegian)

Pageset is now available in Swedish and Norwegian.

Text Based Pageset (English only)

- Added a new column of buttons to the pageset. Now 7x10 instead of 6x10
- Added Message History button to the toolbar
 - Grid-aligned
 - View recently spoken messages
 - You can do the following with messages in the message history:
 - Delete an individual message
 - Send the message to QuickFires, QuickPhrases or the last topic to save them in any message button
 - Copy to the message window in the keyboard
- Keyboard changes:
 - QWERTY keyboard now grid-aligned for new 7x10 keyguard
 - Added phrase prediction and a set of starter phrases and phrase builders
 - Phrase prediction will also use phrase in the message history
 - Added Send To button so messages can be saved to QuickFires, QuickPhrases or the last topic
- QuickFires - Removed the “Casual” and “Formal” pages of phrases
- Updated new page templates to match the 7x10 layouts

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com or www.myTobiiDynavox.com.

Contact Your Reseller

For questions or problems with your product, contact Customer Support at your Tobii Dynavox Reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For our Resellers contact details, see www.TobiiDynavox.com.