

Release Notes

Compass Content

Version 2.5

Release date: 2016-03-30

Updates to Existing Pagesets

- All Access (English)
 - Added option to hide/show individual message window buttons, under Levels>General>Message Window tab.
 - Widened pagebar for improved accessibility.
 - The Slot List Picker page now includes option to browse all word lists or go to the Keyboard, if the desired word is not available in the slot list.
 - T-Series devices only: Added "Send to Text Message" option under Send To>Apps. See the document Tobii Dynavox Compass Text Messaging Quick Start Guide for text messaging details and setup information.
- Navigator (English)
 - Added option to hide/show individual message window buttons under Levels>General>Message Window tab.
 - Widened pagebar for improved accessibility.
 - Added another page of blank buttons to all topics.
 - T-Series devices only: Added "Send to Text Message" option under Send To>Apps. See the document Tobii Dynavox Compass Text Messaging Quick Start Guide for text messaging details and setup information.
- Core First (English)
 - The Slot List Picker page now includes the option to browse all word lists or go to the Keyboard if the desired word is not available in the slot list.
 - T-Series devices only: Added "Send to Text Message" option under Send To>Apps. See the document Tobii Dynavox Compass Text Messaging Quick Start Guide for text messaging details and setup information.
- Master Page (English)
 - T-Series devices only: Added "Send to Text Message" option under Send To>Apps. See the document Tobii Dynavox Compass Text Messaging Quick Start Guide for text messaging details and setup information.
- NavBar (English)
 - T-Series devices only: Added "Send to Text Message" option under Send To>Apps. See the document Tobii Dynavox Compass Text Messaging Quick Start Guide for text messaging details and setup information.
- Stroke & Brain Injury Persona (English)
 - T-Series devices only: Added "Send to Text Message" option under Send To>Apps. See the document Tobii Dynavox Compass Text Messaging Quick Start Guide for text messaging details and setup information.
- New Packages (English)
 - Released Calendar package, which can be imported into existing pagesets. Users can view calendar for the current month and review/add appointments.
 - Released Emergent Games package, which can be imported into existing pagesets. Includes a variety of cause-and-effect games for emergent users.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com or www.myTobiiDynavox.com.

Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

- Release News from Home/School package, which can be imported into existing pagesets. Allows for quick communication between school and family about events that took place at home or school.

Bug Fixes

- Core First (English)
 - Significant improvements to speed of automorphology on core word pages.
 - When selecting "you", "they", "we", or plural nouns, "is" now auto-morphs to "are". "Are" now appears as an option in the All Forms popup for forms of "to be".
 - "Really" has been added as a synonym for "so".
 - All Message Windows are now 'touch-only', so that they cannot be directly selected when using gaze interaction, scanning, or mouse access methods.
 - When disabling the Message Window, the rest of the page now scales to fill the page.
 - Fixed issue with not being able to change videos in Video model pages while in Modify mode.
 - Fixed issue with deleting messages from Message History.
 - Fixed issue with Social Narrative behavior supports requiring each button to be selecting twice when using gaze interaction.
 - Other minor bug fixes
- All Access (English)
 - Fixed issue with topics created using the Create New Topic option showing the incorrect grid size.
 - Fixed issue with core words sometimes disappearing when using the "Grouped" option.
 - Fixed issue with Social Narrative behavior supports requiring each button to be selecting twice when using gaze interaction.
 - Fixed issue with not being able to change videos in Video model pages while in Modify mode.
 - Fixed issue with deleting messages from Message History.
 - The "Choices" topic was previously missing and has been added back in.
 - Other minor bug fixes
- Master Page (English)
 - Fixed issue where tabs on Topic pages were sometimes not selectable when on Level 4 using alternate access methods.
 - Fixed issue with Social Narrative behavior supports requiring each button to be selecting twice when using gaze interaction.
 - Fixed issue with not being able to edit Contingency Map behavior supports in Modify mode.
 - Fixed issue with not being able to change videos in Video model pages while in Modify mode.
 - Added Close button to Slot Picker page so that picker can be closed without selecting a slot filler.
 - Other minor bug fixes
- NavBar (English)
 - Fixed issue with Social Narrative behavior supports requiring each button to be selecting twice when using gaze interaction.
 - Fixed issue with not being able to change videos in Video model pages while in Modify mode.
 - Added Close button to Slot Picker page.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com or www.myTobiiDynavox.com.

Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

- Other minor bug fixes
- Navigator (English)
 - Fixed issue with deleting messages from Message History.
 - Other minor bug fixes
- Stroke & Brain Injury Persona (English)
 - A number of videos were missing from the Levels>Help Videos and have been added back in.
 - Added Close button to Slot Picker page.
- WordPower (English)
 - Fixed "Back" button on Setup Wizard page so that you can select another layout.

Version 2.4

Release date: 2015-09-22

New Pagesets

- French language
 - The All Access - Français and Navigator - Français pagesets are now available.

Version 2.3

Release date: 2015-06-29

New Pagesets

- English language
 - The Core First pageset is now available. It is designed to encourage and develop use of core words while offering optional tools for efficiency and flexibility.
- Dutch language
 - The pagesets All Access — Nederlands and Navigator — Nederlands are now available.
- Danish language
 - The pagesets All Access — Dansk and Navigator — Dansk are now available.

Updates to Existing Pagesets

- All Access (English)
 - Added option for thicker 20-pixel grid spacing.
 - Added Calculator to Dashboard.
- All Access (Norwegian, Spanish, Swedish)
 - Added option for thicker 20-pixel grid spacing.
- Gateway (English)
 - Corrected an issue where "I can do" was being mistaken as abbreviation and changing to "I can does."

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com or www.myTobiiDynavox.com.

Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

- Navigator (English)
 - Added Calculator to Dashboard.
- WordPower (English)
 - Vocabularies updated and enhanced. (All version of WordPower)
 - New scanning vocabularies. (Scanning WordPower 160, Scanning WordPower 100, Scanning Picture WordPower Phrases & Categories)
 - Vocabulary expansion and addition of logical next words. (WordPower 60, WordPower 60 — Phrases & Categories)
 - Updated the Food and Animals categories. (WordPower 60, WordPower 60 — Phrases & Categories)
 - Added the pages Actions A-Z and Describe A-Z. (WordPower 20 — Touch & Scan, WordPower 60, WordPower 60 — Phrases & Categories)

Bug Fixes

- All Access (Norwegian, Swedish)
 - Fixed bug in Core Pages where core words would not reset to the default list after selecting a word prediction or phrase prediction button.
 - Fixed bug where Next Letter Prediction option under Settings>Keyboard was not working.
- Navigator (English)
 - Fixed the bug where the My Phrases tab of Quick Phrases would not scroll to the second page.
- Navigator (Norwegian, Swedish)
 - Fixed bug where Next Letter Prediction option under Settings>Keyboard was not working.

Known Issues

- Core First pageset
 - When toggling between Fitzgerald Key and Keyboard Core organization, the Show Little Words option under Word List Settings fails to update based on grid size.
 - When selecting one of the word form buttons on a Core page, synonyms for that word fail to show, even if the base word has synonyms.

Version 2.2.1

Release date: 2015-04-09

Master Page Pageset (English only)

- There is now only a single Master Page Pageset, as opposed to separate pagesets for each age range. With the addition of the Topic Picker (see below), age-appropriate topics can easily be selected. Message and vocabulary customizations are preserved across layout changes.
- Topics
 - Added Topic Picker/Creator. Access through Levels (Dashboard > Levels > Topics > Select Topics from Library)
 - Disable Alphabetize Topics if you would like to reorder topics on the master page using the Editor.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com or www.myTobiiDynavox.com.

Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

- Topic pages can now toggle between Visual Scene and Grid layouts, preserving changes to messages. Access through Dashboard>Levels>Topics>Page Layout.
- Added 11 new topics: Advocating, Arguing, Asking for Help, Crimes, Dating, Directing My Care, Job Coach, My Feelings, Planning, Presentation and Sexuality
- Supports
 - Social Narrative supports are now editable via the Modify mode. Minor Modify mode editing improvements made to other supports.
- Word Lists
 - Access to separate word list for different ability levels: (levels 1 & 2), context dependent (levels 3 & 4) & independent (Level 5).
 - Change the word lists for a specific level in Settings (Settings > Interface > Age & Ability Level > Ability Level).
 - Note: Customization to words in one ability level will NOT transfer to the others.
 - All word list vocabulary is on a single page to make editing easier.
 - Sexuality words can be enabled/disabled through Levels (Levels > Word Lists > Show Sexuality Words).
- Keyboards
 - Added the non grid-aligned AEIOU keyboard.
 - Added support for Next Letter Prediction
 - Added Phrase Prediction
- Levels
 - Accessible through the Dashboard (see below) or the Settings tool.
 - Level Buttons change the grid layout and ability level for the pageset.
 - Level 1 - 9 button (3x3)
 - Level 2 - 16 button (4x4)
 - Level 2 - 25 button (5x5)
 - Level 2 - 36 button (6x6)
 - Level 2 - 49 button (7x7)
 - Changing the level will reset all level settings, except the page layout, back to the defaults.
 - Added the Dashboard and related apps and features (Accessible from the Master page)
 - Photo Albums
 - Video Models
 - Message History
 - Rating Scale
 - Games: Target practice, Wack- A- Mole, Concentration, Matching, Word Scramble
 - Whiteboard
 - Texting
 - Remotes
 - Apps
 - Calculator - Basic math functions, +, -, X, /, =/-, C, backspace (for current number entry)
 - Volume controls

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com or www.myTobiiDynavox.com.

Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

- Mute toggle
- Levels
- Added Support for Send To - Send messages from the Keyboard, Message History or calculator to a Topic or QuickPhrase button or send to Apps.

Text-Based Pageset (English only)

- Main page can now be set to either topics or the keyboard in Setup Wizard
- Pageset layout is now 7x10 grid size instead of 6x10
- Message History button added to the toolbar
 - View recently spoken messages
 - Delete an individual message
 - Send the message to QuickFires, QuickPhrases or the last topic to save them in any message button
 - Copy to the message window in the keyboard
- Keyboard
 - All keyboard layouts are now grid-aligned for new 7x10 keyguard
 - Added phrase prediction
 - Populated from a set of starter phrases and phrase builders, plus message history
 - Added Send To button so messages can be saved to QuickFires, QuickPhrases or the last topic
- QuickFires - Uses a single page of phrases (Removed the "Casual" and "Formal" options)

All Access - español (Spanish only)

- Pageset has been renamed "All Access - Español"
- Help and intro videos have been translated into Spanish
- Fixed issue with hiding/showing of Support, Scripts, & Topic Words buttons on Topic pages

Version 2.2

Release date: 2015-02-18

All Access (English, Spanish, Swedish, and Norwegian)

- Bug fixes
- Pageset is now available in Swedish and Norwegian.

NavBar Pageset (English only)

- Levels
 - Levels button has been removed from the NavBar, and added to the new Dashboard page (see below).
- Topics
 - Message grid
 - Message buttons can now be edited using Modify mode.
 - Message grid uses swipable pages. Select the grid and use the swipe controller to access additional pages.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com or www.myTobiiDynavox.com.

Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

- Topic pages can now toggle between Visual Scene and Grid layouts, preserving changes to messages. This is accessed through Dashboard>Levels>Topics>Page Layout.
- Option to Enable/Disable Topic Words button or remove Topics from toolbar has been added to the Levels>Topics settings.
- The Vocabulary database (Settings>Vocabulary) is no longer being used for Topic messages. Changes made to Topic messages in the Vocabulary database will NOT be reflected on topic pages.
- Added 11 new topics: Advocating, Arguing, Asking for Help, Crimes, Dating, Directing My Care, Job Coach, My Feelings, Planning, Presentation and Sexuality
- Supports
 - Changes were made to various support to improve compatibility with the Modify mode. Note: The Social Narrative supports are not editable via the Modify mode.
 - Scripts
 - Scripts now swipe if there are multiple pages. Select the grid and use the swipe controller to edit additional pages.
- QuickPhrases
 - Message buttons can now be edited using Modify mode.
 - QuickPhrases Grid Layout is now independent of the Topic Grid Layout. Access through Dashboard>Levels>-QuickPhrases>Grid Layout
 - My Phrases tab is now visible at all levels
- Word Lists
 - Folders for sub-word lists are now at the end of the current word list, instead of the beginning.
- Core
 - Core Pages have been significantly revised.
 - Sub-categories of core ("verbs", "adjectives", etc) have been removed
 - Core Pages now have only a Main Tab for all the core words, and a Personal tab for any personalized words the user wants to add.
 - 7 Grid Layouts are available: 4x5, 5x6, 6x7, 7x8, 8x10, 10x10, 12x13
 - At 4x5 layout, you have a choice of how many words to expose (6/9/12/20).
 - At higher layouts, all words are shown.
 - Color coding option has been added
- Keyboards
 - Added phrase prediction button to all of the prediction keyboards
 - Added "AEIOU" (without word prediction) option
 - Added Next Letter Prediction toggle button to all keyboards
- Levels menus
 - Added General setting dialog, which contains settings that apply to all across all the different types of pages
 - New dialogues added for Dashboard and Message History
- A new Dashboard page has been added to the toolbar. This contains some of the functions that were previous located elsewhere, as well as some new functions. Items found in the Dashboard are:
- New Items:

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com or www.myTobiiDynavox.com.

Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

- Message History – shows past messages. Includes only novel sentences constructed in Core or Keyboard pages.
- Games – these are set of games provided for fun and for practice targeting buttons.
- Calculator – a simple calculator for basic arithmetic functions.
- Items Re-located to Dashboard:
 - Photo Albums – these replace the Photo Stories, and have an updated appearance.
 - Video Models
 - Rating Scale
 - Whiteboard
 - Remotes
 - Apps
 - Volume controls
 - Mute
 - Levels
- Send To button added to most message windows. Allows send of message window text to another page to be stored on a button, to the Keyboard page to be edited, or to an outside App.
- Slot Picker page was revised

Stroke & Brain Injury Persona Pageset (English only)

- Levels
 - Levels button has been removed from the Navbar, and added to the new Dashboard page (see below).
- Topics
 - Message grid
 - Message buttons can now be edited using Modify mode.
 - Message grid uses swipable pages. Select the grid and use the swipe controller to access additional pages.
 - Option to Enable/Disable Topic Words button or remove Topics from toolbar has been added to the Levels>Topics settings.
 - The Vocabulary database (Settings>Vocabulary) is no longer being used for Topic messages. Changes made to Topic messages in the Vocabulary database will NOT be reflected on topic pages.
 - Added 9 new topics: Advocating , Arguing/Negotiations , Asking for Help , Choices , Crimes , Dating , Planning , Presentations , Football
- QuickPhrases
 - Message buttons can now be edited using Modify mode.
 - QuickPhrases Grid Layout is now independent of the Topic Grid Layout. Access through Dashboard>Levels>-QuickPhrases>Grid Layout
 - My Phrases tab is now visible at all levels
- Word Lists
 - Folders for sub-word lists are now at the end of the current word list, instead of the beginning.
- Keyboards
 - Added phrase prediction button to all of the prediction keyboards

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com or www.myTobiiDynavox.com.

Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

- Added "AEIOU" (without word prediction) option
- Added Next Letter Prediction toggle button to all keyboards
- Levels menus
 - Added General setting dialog, which contains settings that apply to all across all the different types of pages
 - New dialogues added for Dashboard and Message History
- A new Dashboard page has been added to the toolbar. This contains some of the functions that were previous located elsewhere, as well as some new functions. Items found in the Dashboard are:
 - New Items:
 - Message History – shows past messages. Includes only novel sentences constructed in Core or Keyboard pages.
 - Games – these are set of games provided for fun and for practice targeting buttons.
 - Calculator – a simple calculator for basic arithmetic functions.
 - Items Re-located to Dashboard:
 - Photo Albums – these replace the Photo Stories, and have an updated appearance.
 - Rating Scale
 - Whiteboard
 - Remotes
 - Apps
 - Volume controls
 - Mute
 - Levels
 - Send To button added to most message windows. Allows send of message window text to another page to be stored on a button, to the Keyboard page to be edited, or to an outside App.
 - Slot Picker page was revised

Navigator Pageset (English, Spanish, Swedish, and Norwegian)

Pageset is now available in Swedish and Norwegian.

Text Based Pageset (English only)

- Added a new column of buttons to the pageset. Now 7x10 instead of 6x10
- Added Message History button to the toolbar
 - Grid-aligned
 - View recently spoken messages
 - You can do the following with messages in the message history:
 - Delete an individual message
 - Send the message to QuickFires, QuickPhrases or the last topic to save them in any message button
 - Copy to the message window in the keyboard
- Keyboard changes:
 - QWERTY keyboard now grid-aligned for new 7x10 keyguard
 - Added phrase prediction and a set of starter phrases and phrase builders

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com or www.myTobiiDynavox.com.

Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

- Phrase prediction will also use phrase in the message history
- Added Send To button so messages can be saved to QuickFires, QuickPhrases or the last topic
- QuickFires - Removed the “Casual” and “Formal” pages of phrases
- Updated new page templates to match the 7x10 layouts

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com or www.myTobiiDynavox.com.

Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact